


## NEOMASK Fitting/Assembly Instructions:

SKU: SG-NEO-SINGLE

1. Place the mask to your face and position it comfortably between the nose-bridge and chin.
2. Secure the mask by contracting the two Velcro pads at rear of head.
3. Shape the nose clip to ensure a good seal.
4. To check the fit, cup both hands over the face and exhale vigorously. If air flows around the edges, increase the tension.
5. By rotating the Velcro pads the tension at the top and bottom edge of the mask can be adjusted.

OPTIONAL:

The attached ear plugs can be used to muffle loud noise.

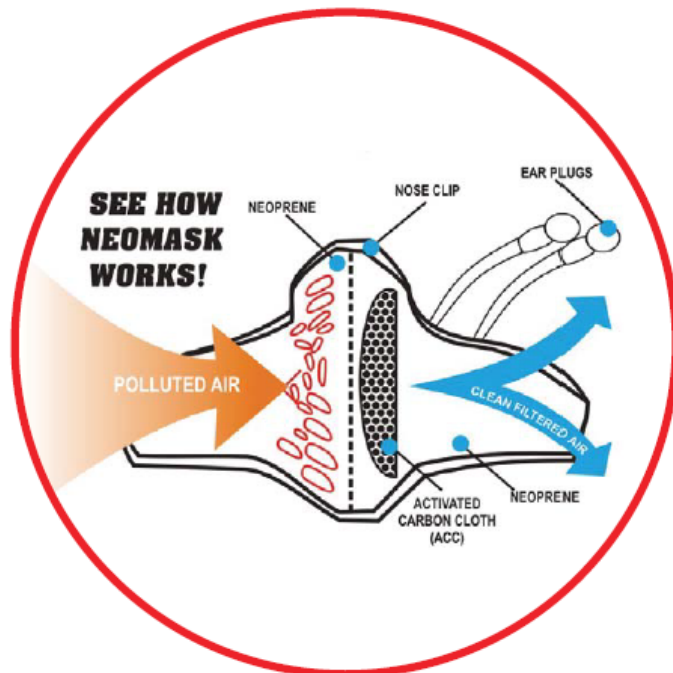

**NEOMASK Fitting/Assembly Instructions:**

SKU: SG-NEO-Filters

1. Unsnap, remove core filtration system from NEOMASK.
2. Remove non-woven filter from Activated Carbon Cloth and NEOMASK.
3. Discard non-woven filter.
4. Place new non-woven filter on core filtration system.
5. Snap back onto NEOMASK.