

# Other Genuine Big John Products



## BIG JOHN COMMODE CHAIR WITH BUCKET

1,000 lb. capacity  
Versatile use:  
Bedside or over toilets

## BIG JOHN TOILET SUPPORTS

### REGULAR & SHORT SWIVEL

The secure solution for supporting wall mounted toilets...



The Short Swivel Support is a great alternative when the Regular Support doesn't fit.



Top of Short Support Swivels



Increases capacity to 1,000 lbs.  
Protect yourself from liability



Support Stop

Part #1-SS

## BIG JOHN TOILET SEAT

The new era of stability, comfort and safety

The Big John toilet seat in comparison with a regular sized seat.  
**75% Larger Sitting Area**



### STANDARD SEAT

800 lbs. Capacity  
1 1/2" Rise  
ABS Hinges  
19" wide

### ORIGINAL SEAT

Open or Closed Front  
1,200 lbs. Capacity  
2 1/2" Rise  
Stainless Steel Hinges  
19" wide

### CLASSIC SEAT

Available in Black or White  
1,000 lbs. Capacity  
2 1/2" Rise  
Stainless Steel Hinges  
14" wide



## BIG JOHN HYGIENIC SPRAYER



For everyone's personal hygiene



Very easy to install  
New T-connector installs directly to tank.

## BIG JOHN BARIATRIC STEP STOOL



700 lb. Capacity

Weights only 3.13 lbs • Non-Slip Feet  
Folds for Easy Storage • Durable Construction

# The Big John Original



19" wide from Side to Side



Big John Products 8533 Canoga Ave. Suite D, Canoga Park, CA 91304

**TOLL FREE: 866-366-0669**

ADA Compliant Registration Number 4.23 ADAAG

Meets or exceeds

ASME / ANSI Z124.5

CSA

IAPMO / UPC



**MADE IN THE U.S.A.**



# Installation Instructions

**MADE IN THE U.S.A.**

[www.bigjohnproducts.com](http://www.bigjohnproducts.com)




# 9 Easy Step-by-Step Instructions for Installing the Big John Toilet Seat

## Step 1



Screw flanges all the way up so that there are no threads showing

## Step 2

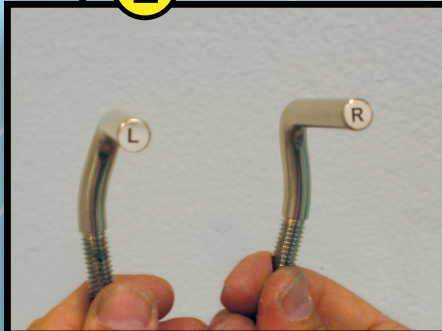

Hinges are marked right and left

## Step 3



Insert the Left Hinge through the seat and lid on the Left side\*

## Step 4



Insert Right Hinge\*

## Step 5




Put on the washers

## Step 6



Mount on toilet and adjust flanges so bumpers are flat **See warning below\***

## Step 7

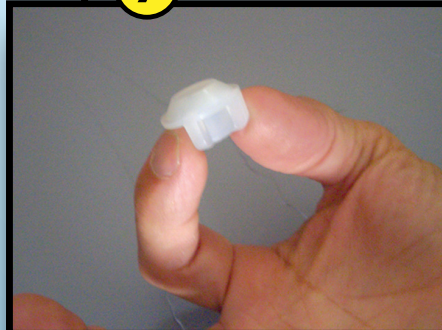

Be sure to put on the nuts with the beveled side up

## Step 8



Finger tighten the nuts...make sure that the seat is mounted properly

## Step 9



Secure the seat using pliers...  
DO NOT OVER TIGHTEN.

\*Left / Right Hinges - Refers to the left and right mounting holes on the toilet when facing it.

\*View installation video at ..... [www.bigjohnproducts.com](http://www.bigjohnproducts.com)



**WARNING: ADJUST FLANGES SO THE BUMPERS SIT FLAT ON THE TOILET BEFORE TIGHTENING NUTS. Cracking or breaking around the hinges is a result of improper installation and IS NOT covered by the warranty!**