
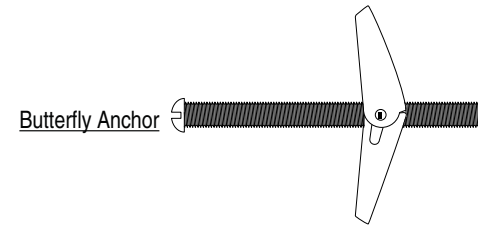
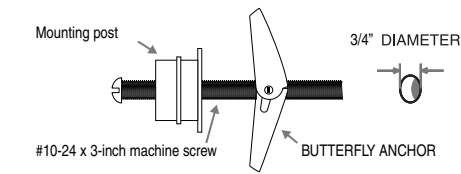
ANCHOR INSTRUCTIONS FOR NON-STUD MOUNT


1. Drill 3/4-inch hole into wallboard on non-stud side only. Insert the #10-24 x 3-inch machine screw supplied with the anchor through the mounting post and into the butterfly anchor.

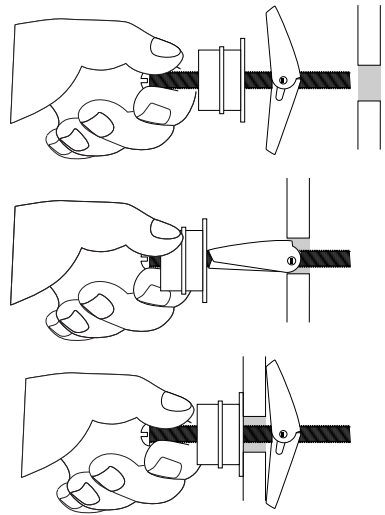
2. While holding the butterfly anchor and the mounting post, slide the anchor through the drilled hole until the wings open. Tighten the #10-24 x 3-inch machine screw until the mounting post is flush with the wall.

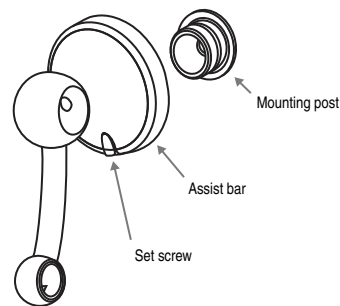
3. Fasten Assist Bar post to mounting post with set screw.

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

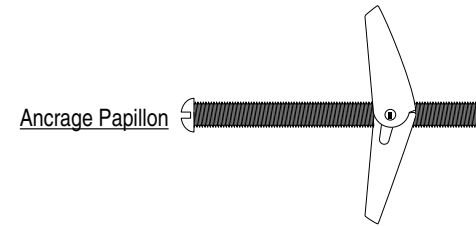
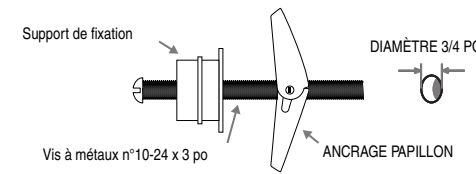
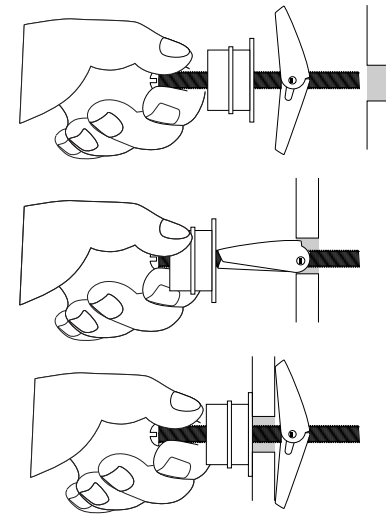
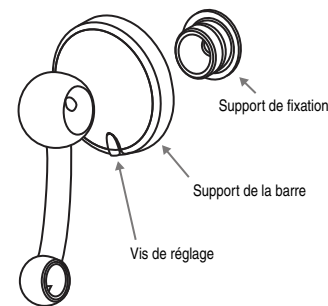
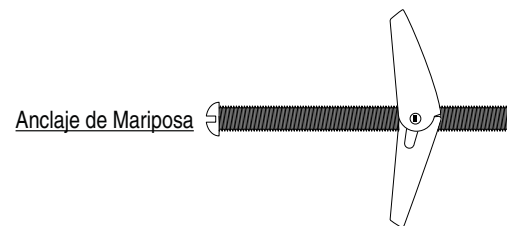
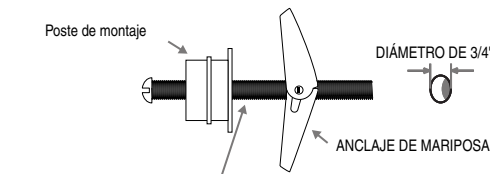
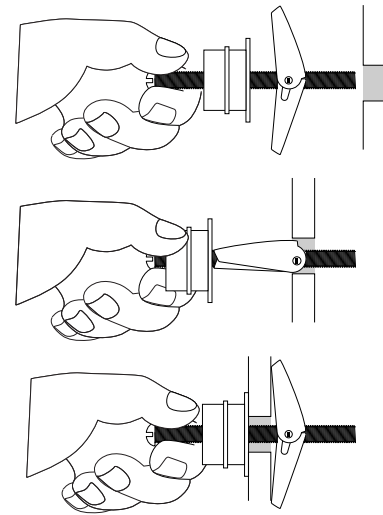
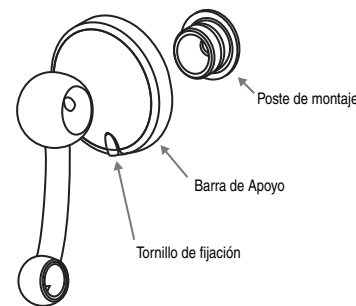
Residential Assist Bar INSTALLATION, OPERATING AND CARE INSTRUCTIONS

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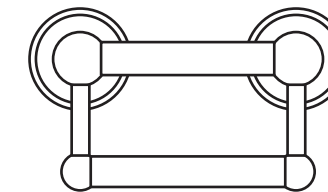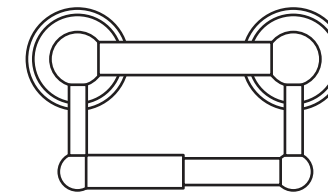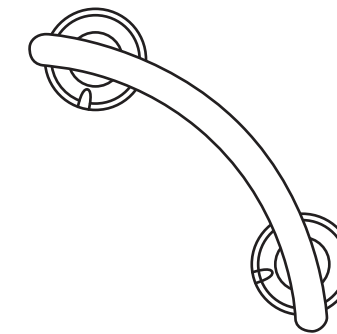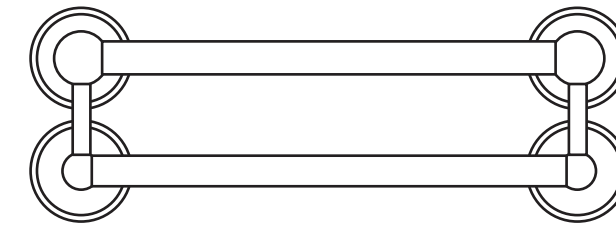Assist Bar - Hidden In Plain Sight Series

INSTALLATION, OPERATING AND CARE INSTRUCTIONS:

SAFETY WARNINGS:

- **DO NOT** install this equipment without first reading and understanding this instruction sheet. If you are unable to understand these Warnings and Instructions, contact a health care professional, dealer or technical personnel for assistance before attempting to install this equipment - otherwise, injury may occur.
- After ANY adjustments, repair or service and BEFORE use, make sure that all attaching hardware is tightened **securely**.
- Assist Bars add security for bath and commode areas if attached as instructed. Location, size, and angle of bars should be determined by a qualified professional to provide optimum safety and function.
- Use **EXTREME** caution on wet tub or floor surfaces.
- Users with limited physical capabilities should be supervised or assisted in bath and commode areas, even when using Assist Bars.
- This Assist Bar can provide support and increased stability for an individual weighing up to **300 pounds**. Assist Bars are NOT designed to support the total weight of an individual. **Use the bar for assistance ONLY.**
- Be sure Assist Bars are correctly and securely installed. NEVER use Assist Bars if any looseness is noted.

TOOLS REQUIRED:

- Safety Glasses
- Pencil
- 3/8-inch Power Drill
- 3/16-inch Drill Bit (to drill pilot hole for installation of Lag screw into wood wall stud)
- 5/16-inch Carbide Tip Drill Bit (if drilling through ceramic tile)
- 7/16-inch Hex Socket with 3/8-inch Drive Ratchet
- Stud Finder
- 3mm Hex Key

PARTS SUPPLIED:

- 1/4-inch x 2-1/2 inch Hex head Lag Screws

IMPORTANT:

The recommended mounting method for assist bars is to mount BOTH ends of the bar into WOOD wall studs. For help locating wood wall studs, please visit our website at www.libertyhardware.com.

BEFORE INSTALLING ASSIST BAR:

1. Remove the Assist Bar and parts from the package.
2. Locate wood wall studs.
3. Place Assist Bar on wood wall stud at desired position.
4. Lightly mark the wall where the notch is on the flange, which will be on the outside and lightly trace a circle around the flange base for alignment in a later step (see Figure 1A).
5. Remove set screw from the post using a 3mm Hex Key.
6. Remove the flange from the bar (see Figure 1B).
7. Place flange on the wall. Be sure to align the flange with the marked notch and center flange within the circle drawn in step 4.
8. Mark the mounting hole location in the center of the flange. If installing over ceramic tiles, place masking tape on the mounting area before marking the hole location.

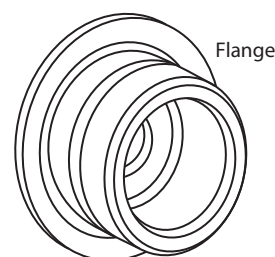
Figure 1A


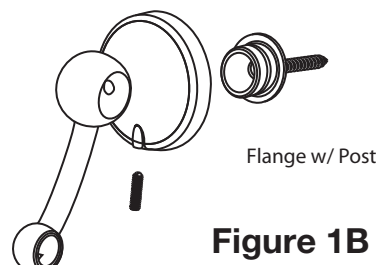
Figure 1B

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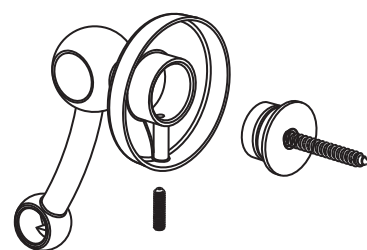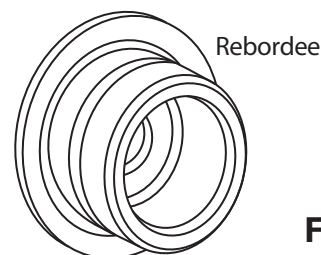
Figura 1A

Figura 1B

BOTH ends of residential assist bar into WOOD wall studs

⚠ WARNING

ASSIST BAR MAY NOT PROVIDE DESIGNED AMOUNT OF SUPPORT UNLESS THESE INSTALLATION INSTRUCTIONS ARE STRICTLY FOLLOWED.

For proper installation, both ends of Assist Bar MUST be positioned over a wall stud. For solid support, both fasteners MUST go through the wallboard or tile and be tightly secured into the wall stud. Assist Bar may be installed vertically, horizontally or at an angle, so the mounting screw on both ends of the Assist Bar is secured into a wall stud (See Figures 2 & 3). Locate the wall studs behind the wall before beginning installation (a stud finder can be obtained from a hardware store).

⚠ WARNING

THE FOLLOWING STEPS REQUIRE DRILLING INTO THE WALL.

When drilling into the wall, exercise care to avoid any electrical wiring or plumbing that may be located behind the wall. Damaged electrical wiring can cause electrical shock and/or fire. Since older homes do not always fall in line with current housing codes and requirements, know where internal wall wiring is located so that no wires will interfere with your installation. **PROPER INSTALLATION IS EXTREMELY IMPORTANT. IF IN DOUBT, INSTALLATION SHOULD BE DONE BY A QUALIFIED PROFESSIONAL.**

TO INSTALL INTO WOOD WALL STUDS:

9. Put on safety glasses before starting.
10. Use the 3/8-inch power drill and 3/16-inch drill bit to drill a hole into the wall and wood wall studs at the marked position. **NOTE: If installing Assist Bar over ceramic tiles, the 5/16-inch carbide tip drill bit will be needed for drilling through the tile. Drill clearance hole through the ceramic tile by using a 5/16-inch carbide tip drill bit. Once a hole has been drilled through the tile, use the 3/16-inch drill bit to drill a pilot hole in the wood wall stud.**
11. Attach flange to wall with the 1/4-inch Lag screw using the 7/16-inch 3/8-inch drive socket and ratchet. **DO NOT COMPLETELY TIGHTEN**
12. Align the notch on the flange to the mark on the wall. Tighten screw. Bar should be firm against wall, but not so tight that it could damage wall or component parts.
13. Place bar onto post.
14. Rotate the assist bar on the post to align the opposite flange with the second wood wall stud.
15. Lightly trace around the second flange with pencil.
16. Lightly mark the wall where the notch is on the flange, which will be on the outside.
17. Remove set screw from the post using the 3mm Hex Key provided.
18. Remove the flange from the bar.
19. Place flange on the wall centered within the traced circle and mark the mounting hole location in the center of the second flange.
20. Repeat Steps 10-13.
21. Install set screws. Securely tighten.

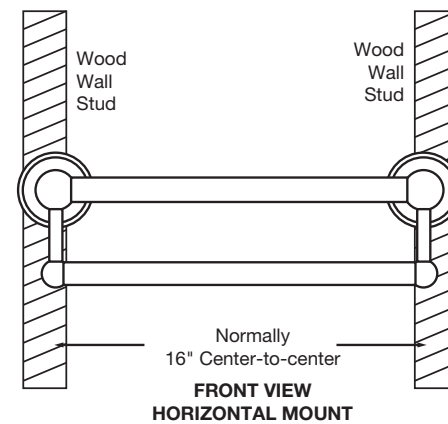
Figure 2

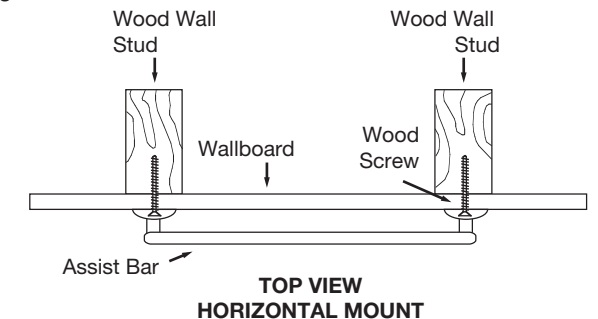
Figure 3A

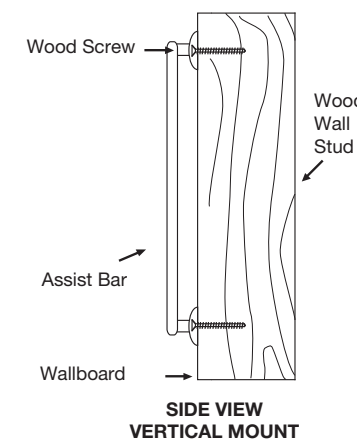
Figure 3B

USING RESIDENTIAL ASSIST BAR:

1. Read, understand and follow the information in the **SAFETY WARNINGS** section of this instruction booklet. Liberty Hardware Manufacturing Corporation shall have no liability for any damage or injury caused by improper installation or use of this Residential Assist Bar.
2. Grasp the Residential Assist Bar when exiting or entering the bathtub or when sitting down or rising from the commode.
3. Use EXTREME caution on wet tub or floor surfaces.
4. **DO NOT** put entire weight on Residential Assist Bar. Residential Assist Bar is rated for persons weighing up to **300 pounds** when properly installed.
5. **DO NOT** stand, sit or step on Residential Assist Bar. **DO NOT** use Residential Assist Bar as a step or platform to reach area above the tub. **Use the bar for assistance ONLY.**

CARE AND MAINTENANCE:

1. Use of mild soap and water or a non-abrasive cleaner is recommended for cleaning.
2. Periodically inspect the Residential Assist Bar for damage or looseness. **DO NOT** use if any damage or looseness exists. Contact your authorized dealer for assistance.
3. **SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE.**