



Wheelchair & Electric Scooter Ramp: Instructions



Installation:

- Ensure the top of the ramp is pressed firmly against the step or platform before using.
- To prevent the ramp from moving; adhere to ground using pre-drilled mounting holes with steel pins (not included).

Safety:

- Read and understand instructions before using this ramp.
- Use caution and care when using the ramp.
- A caregiver should always be present.
- DO NOT USE this ramp if step height exceeds 9"

Going Up the Ramp:

- Remove anything that might cause the device to tip backwards.
- Fasten the seat belt and confirm anti-tip wheels are functional.
- Center the chair or scooter at the base of the ramp.
- If using a powered vehicle proceed at the lowest speed.
- As you start up the ramp, focus on going forward.

Going Down the Ramp:

- Gravity will influence your device, so proceed with caution.
- With seat belt fastened and chair centered, power up and go slowly.
- A chest strap used with the seat belt will help stabilize the upper body and keep you from leaning forward.
- As you approach the landing, ease up on the speed and come to a slow stop.

Questions or Concerns? Visit Stalwartproducts.com