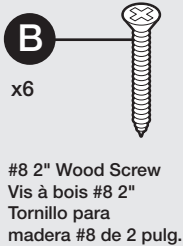
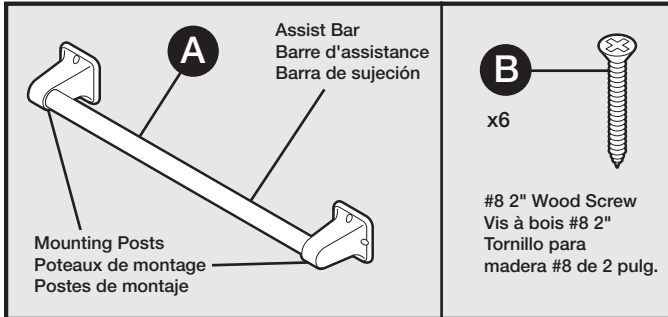
see what Delta can do™

# Assist Bar / Barre d'assistance / Barra de sujeción

## PACKAGE CONTENTS CONTENU DE L'EMBALLAGE CONTENIDO DEL PAQUETE

## IMPORTANT SAFETY INFORMATION

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Read all instructions and warnings prior to beginning installation of this Assist Bar. Save this instruction booklet for future reference.

**⚠ WARNING** To reduce the likelihood of injury, thoroughly read and understand all safety information and instructions provided in this instruction booklet. If you are unable to understand any of the safety information and/or installation instructions, contact a professional for assistance before attempting to install.

## SAFETY INFORMATION FOR INSTALLATION

**⚠ WARNING ELECTRICAL SHOCK HAZARD! CUTTING OR DRILLING INTO ELECTRICAL CORDS OR CABLES CAN CAUSE DEATH, SERIOUS PERSONAL INJURY AND FIRE! ALWAYS** make certain area behind mounting surface is free of electrical wires and cables before drilling or installing fasteners. Proper installation is extremely important. If in doubt, installation should be done by a qualified professional.

**⚠ WARNING** To avoid injury to eyes, wear safety glasses during installation of Assist Bar.

**⚠ WARNING** To help reduce the likelihood of serious injury, read and comply with all safety information and instructions. Assist Bar may not provide the designed amount of support unless these installation instructions are strictly followed.

- Assist Bar was designed and tested for interior bathroom use. Assist Bars add security for bath and commode areas if attached as instructed. Location, size and angle of bars should be determined by a qualified professional to provide optimum safety and function. This Assist Bar can be installed in the bathing area. For placement recommendations, please visit [www.LibertyHardware.com](http://www.LibertyHardware.com).
- Both ends **MUST** be secured to a wall stud. If that's not possible, use only a Delta Hollow Wall Anchor (sold separately), DF548 (pictured to the left), or other wall anchor designed for Assist Bar installation is used. **DO NOT** use plastic anchors, toggle bolts, or similar devices. If a wall anchor must be used, the safest practice is to have one side of the assist bar attached to a wall stud.
- At least two (2) of the three (3) screws on each end of the Assist Bar **MUST** go through the wallboard or tile and be tightly secured into the wall stud or wood blocking, unless a wall anchor is used.
- If using the Delta Hollow Wall Anchor (DF548), three anchors per side **MUST** be used.

## SAFETY INFORMATION FOR USE OF ASSIST BAR

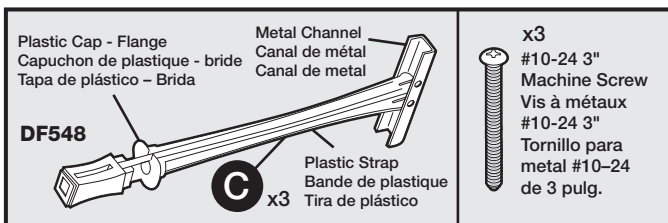
**⚠ WARNING** This Assist Bar can provide support and increased stability for an individual weighing up to 300 pounds. Assist bars are **NOT** designed to support the total weight of an individual. To reduce likelihood of falling serious injury due to failure of Assist Bar:

- Use Assist Bar for assistance **ONLY**.
- **DO NOT** stand, sit, or step on Assist Bar.
- **DO NOT** use Assist bar as a step or platform to reach area above the tub.


**⚠ WARNING** Falls can cause a serious injury. To reduce that risk:

- Grasp the Assist Bar when exiting or entering the bathtub or when sitting down or rising from the commode.
- Use caution on wet tub or floor surfaces. Wet tub or floor surfaces increase the risk of a fall.
- Supervise and/or assist users with limited physical capabilities in bath and commode areas, even when using Assist Bars.
- **DO NOT** use Assist Bar if any looseness is noted. After **ANY** adjustments, repair or service and **BEFORE** use, make sure that all attaching hardware is securely tightened.

## WALL ANCHORS (SOLD SEPARATELY) ANCRES MURAUX (VENDUS SÉPARÉMENT) ANCLAJES PARA PARED (SE VENDEN POR SEPARADO)

## REQUIRED TOOLS / OUTILS REQUIS / HERRAMIENTAS NECESARIAS
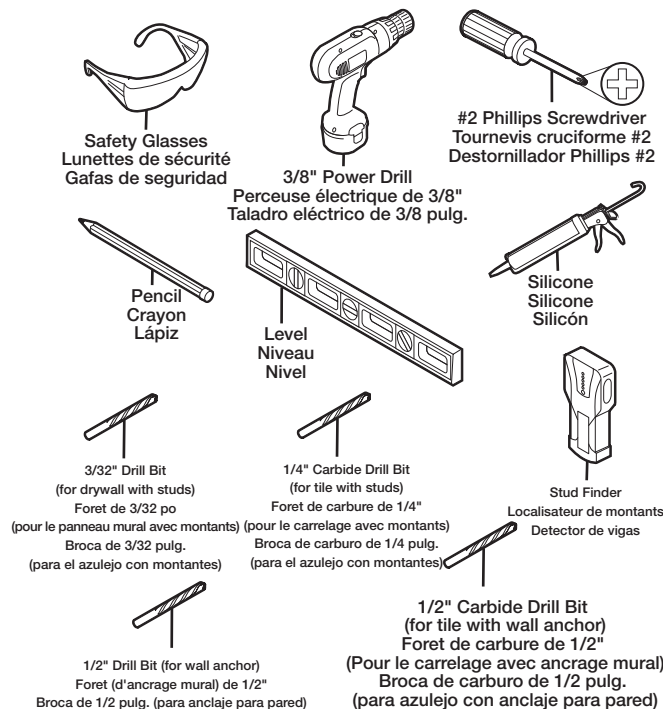
## CARE AND MAINTENANCE

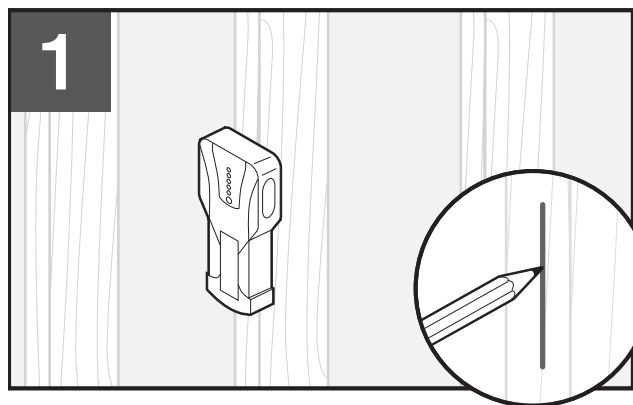
- Use of mild soap and water **ONLY** is recommended for cleaning.
- Periodically inspect the assist bar for damage or looseness.

**BEFORE YOU BEGIN / AVANT DE COMMENCER / ANTES DE EMPEZAR**

Remove contents from package. Read all safety information and instruction steps from page one and instruction steps.

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
Locate and mark wall studs in desired mounting area. If wood studs cannot be located or if studs are metal, refer to Step 5a, Wall Anchor Method.

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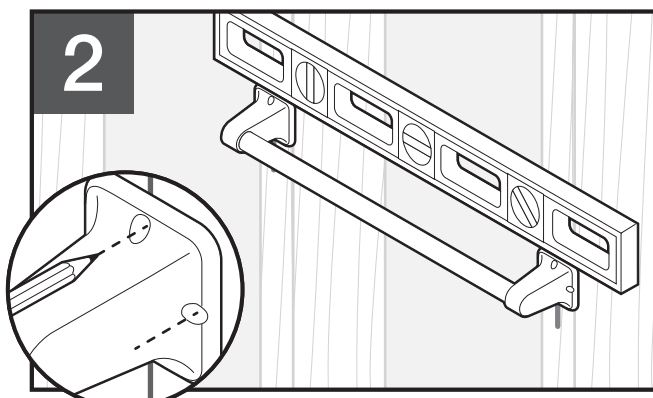
Position assist bar over stud marks and mark locations for mounting holes. Two (2) of the three (3) screws MUST be mounted into stud.

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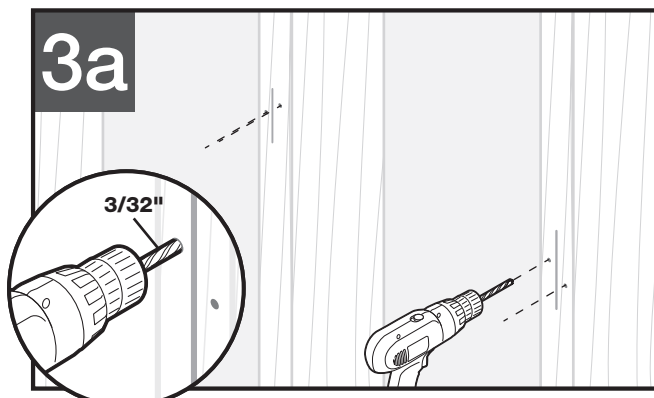
Drill pilot holes into studs with a 3/32" drill bit. If installing into tile, use a 1/4" carbide drill bit.

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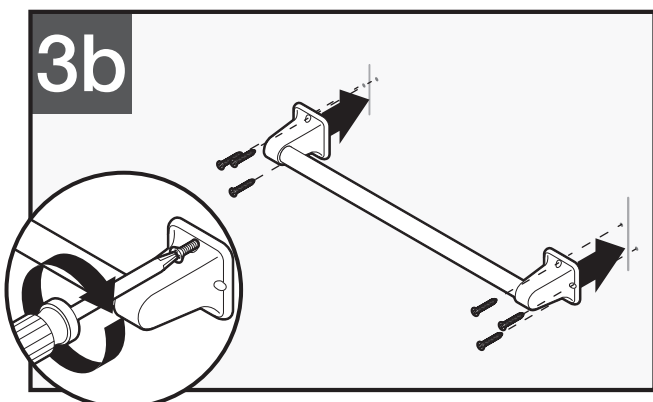Attach both ends of assist bar to the studs using two (2) of the three (3) provided screws. Securely tighten all screws with a screw driver.

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**NOTICE / AVIS / AVISO**

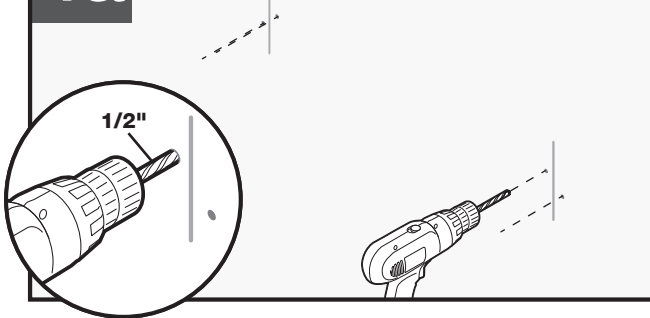
When installing assist bar in shower or bathing area, apply a bead of silicone sealant around the back of the mounting posts before inserting and tightening screws. This will help prevent water from leaking behind wall and damaging wall board.

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4a

Wall Anchor Method / Méthode d'ancrage mural / Método de anclaje para pared


Drill three holes in desired location with a 1/2" drill bit. If installing into tile, use 1/2" carbide drill bit.

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4b

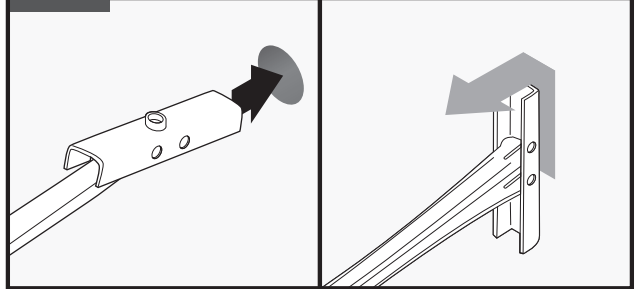Wall Anchor Method / Méthode d'ancrage mural / Método de anclaje para pared


Hold metal channel flat against plastic straps and insert into hole in wall. Hold ends of plastic straps and pull forward until metal channel is flush with wall.

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4c

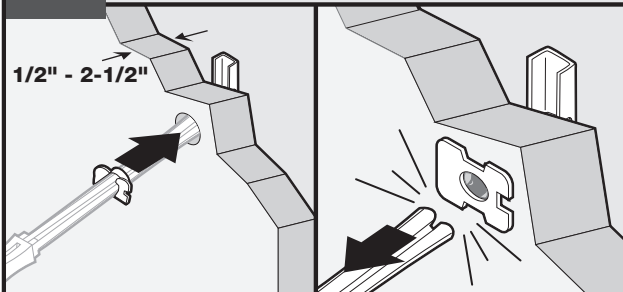Wall Anchor Method / Méthode d'ancrage mural / Método de anclaje para pared


Slide plastic cap along straps until flange of cap is flush against wall. Move straps from side to side, snapping off straps.

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4d

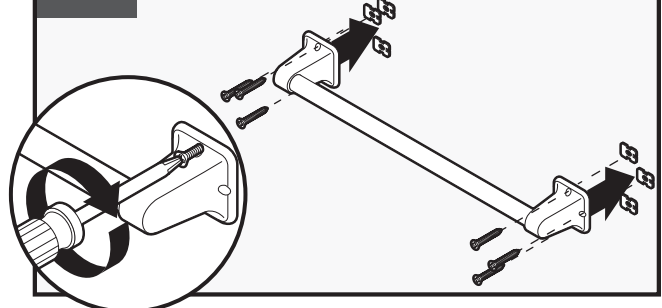Wall Anchor Method / Méthode d'ancrage mural / Método de anclaje para pared

Attach assist bar mounting posts to the anchor using provided 3" machine screws. Securely tighten all screws with a screwdriver.

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**NOTICE / AVIS / AVISO**

*Installer: Leave these instructions with the user. When installing assist bar in shower or bathing area, apply a bead of silicone sealant around the back of the mounting posts before inserting and tightening screws. This will help prevent water from leaking behind wall and damaging wall board.*

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