Moen Home Care is a leader in grab bars, offering top-quality products that are attractive and innovative, with performance features like a 250lb. weight capacity.

The entire Moen Home Care family of products offers people new options for independent living by providing outstanding comfort, convenience and style.

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**Note: Not to be used by persons weighing more than 250lbs.**

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**Integrated accessories are not designed to support weight leverage.**

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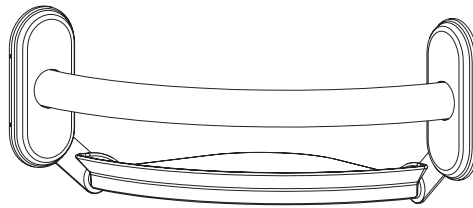
### Components included / Pièces incluses / Componentes incluidos

#### Designer Grab Bar with Corner Shelf

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Requires a 3 1/2" space behind the wall and works in the following substrates

- 1/2" or 5/8" Dry Wall
- 1/2" or 5/8" Dry Wall with Tile
- 1/2" or 5/8" Dry Wall with Marble
- 1/8" or 1/4" Fiberglass Tub Surround

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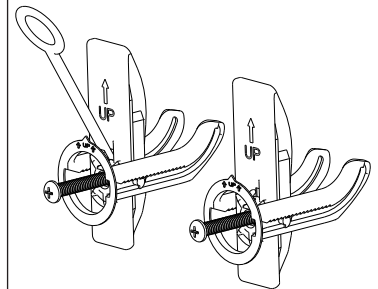
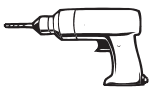
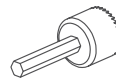
**NOTE:** These instructions are for installation using provided SecureMount Anchors for mounting into drywall, not studs. This product can be used mounted into wood studs if the studs are available in the desired mounting location. If mounting into a wood stud, use four (4) #10 x 2" long stainless cap head wood screws (screws not provided).

**Not to be used by persons weight more than 250 lbs.**

**Not designed to support weight leverage**

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## These instructions are for installation using provided SecureMount™ Anchors (Drywall mounting – off stud location)

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1

**Warning: Use stud finder to avoid drilling into studs**

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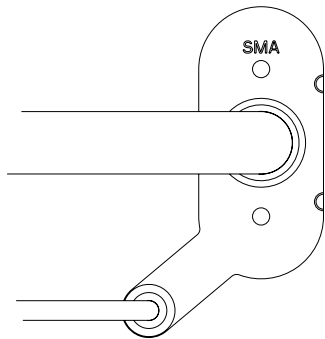
Snap off decorative covers from mounting flange. Place product in desired mounting location. Make sure product is flush against both walls. Use level to check horizontal alignment

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2

Use pencil to mark top hole of the flanges that are marked with "SMA"

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3

We strongly encourage you to determine if plumbing, electrical or other obstructions are contained behind the wall surface prior to drilling.

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4

Use 1.25" hole saw and drill hole at marked locations

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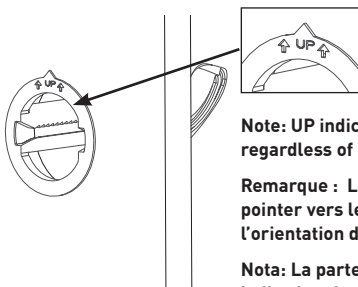
5

Insert SecureMount guide into drilled hole as illustrated – outside circle line on drawing needs to be taken out

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**Note: UP indicator must always point up regardless of grab bar orientation.**

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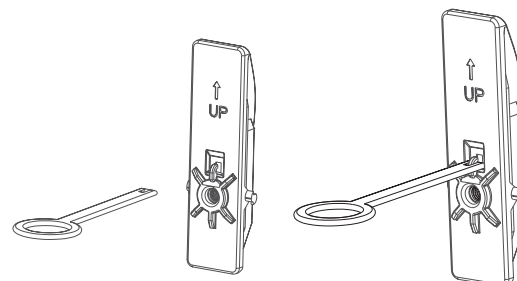
6

Connect pull tab to SecureMount Anchor by pushing down on the tab

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

**7**

With pull tab, insert the anchor all the way into the hole in the wall. Outside circle line on left drawing needs to be taken out

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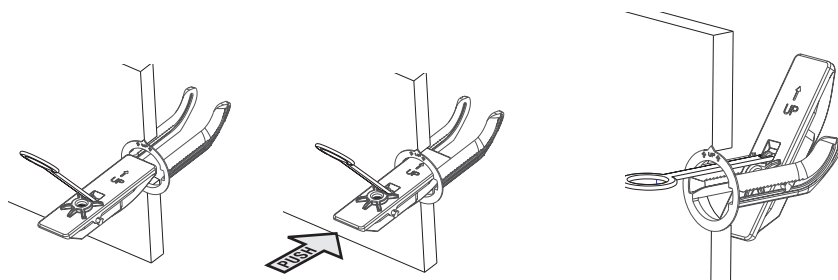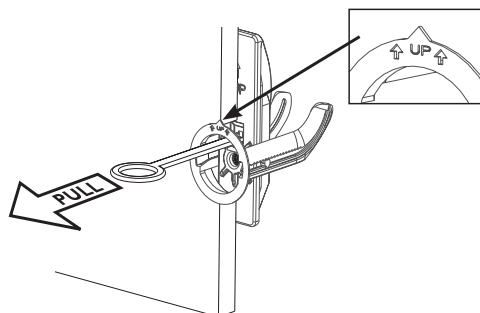**8**

Pull back on the pull tab until the anchor is tight against the wall. Disconnect the pull tab from the anchor by pulling up.

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
**9**

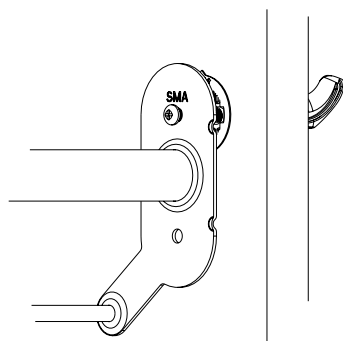Place shelf over mounting holes and secure and tighten screws (insert image- js to provide)

Placez la tablette au-dessus des trous et serrez les vis (insert image- js to provide).

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
**10**

Snap decorative covers over mounting flange

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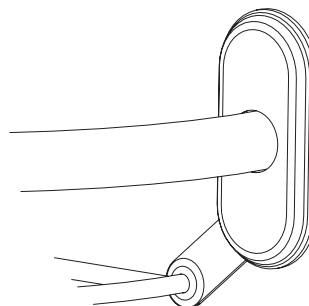
**11**

Place tray on wire shelf and snap into place

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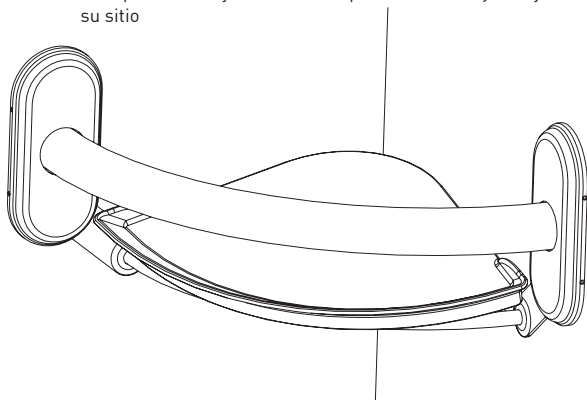
**12**

Ensure product is securely installed before use

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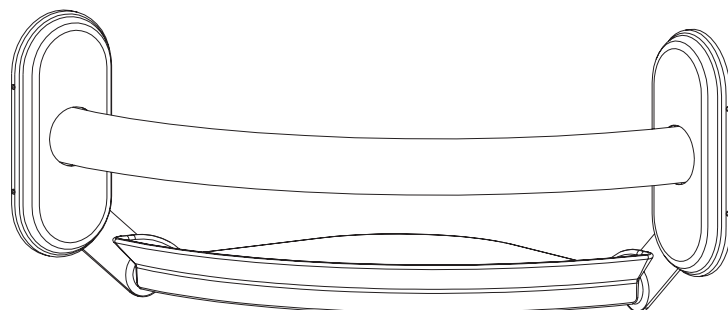**WARNING:**

TO PREVENT SERIOUS OR FATAL INJURY: FOR USE ONLY BY PERSONS WEIGHING LESS THAN 250 POUNDS. Consult a healthcare professional for advice and instructions specific to your needs before use. Contact CSI immediately at 1-800-882-0116 if any components are missing or damaged. Install in accordance with CSI's instructions. Do not modify this product in any way. Check grab bar periodically after installation to be sure all screws are secure. FOR QUESTIONS OR ASSISTANCE WITH INSTALLATION CONTACT CSI AT 1-800-882-0116.

**LIMITED LIFETIME WARRANTY:**

If this product fails due to materials or workmanship at any time during the life of the product, Creative Specialties International (CSI), a division of Moen Incorporated, will replace it free of charge, postage-paid. Simply contact CSI at 1-800-882-0116 for replacement information. This warranty does not cover products which have been abused, altered, damaged, misused, cut or worn. TO THE EXTENT PERMITTED BY LAW, CSI DISCLAIMS ALL OTHER IMPLIED OR EXPRESS WARRANTIES INCLUDING ALL WARRANTIES OF MERCHANTABILITY AND/OR FITNESS FOR A PARTICULAR PURPOSE.

**LIMITATION OF LIABILITY:**

THIS WARRANTY IS YOUR SOLE REMEDY AND CSI SHALL NOT BE LIABLE FOR ANY DAMAGES, WHETHER DIRECT, INDIRECT, INCIDENTAL, SPECIAL, CONSEQUENTIAL, EXEMPLARY, OR OTHERWISE, INCLUDING LOST REVENUES AND LOST PROFITS, ARISING OUT OF ANY THEORY OR RECOVERY, INCLUDING STATUTORY, CONTRACT OR TORT. NOTWITHSTANDING THE TERM OF ANY LIMITED OR IMPLIED WARRANTY, OR IN THE EVENT THAT ANY LIMITED WARRANTY FAILS OF ITS ESSENTIAL PURPOSE, IN NO EVENT WILL CSI'S ENTIRE LIABILITY EXCEED THE PURCHASE PRICE OF THIS PRODUCT. SOME STATES AND PROVINCES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES SO THE ABOVE LIMITATIONS OR EXCLUSIONS MAY NOT APPLY TO YOU. THIS LIMITED WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE RIGHTS WHICH VARY FROM STATE TO STATE AND PROVINCE TO PROVINCE.

For additional warranty information, visit [www.moen.com/homecare](http://www.moen.com/homecare)

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Home Care and Creative Specialties International are registered trademarks of Moen Incorporated ©2012  
[www.moen.com/homecare](http://www.moen.com/homecare)

Creative Specialties is a division of Moen Incorporated. © 2012  
Creative Specialties est une division de Moen incorporée. © 2012  
Creative Specialties es una división de Moen Incorporated. © 2012